


# PickCap Impression Kit

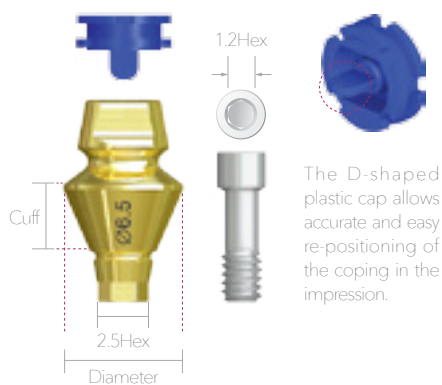

Most accurate closed-type pick-up impression technique.




Connect the correct size of copings onto the placed implants.



Obtain a bite registration.



Attach the plastic caps onto the copings.

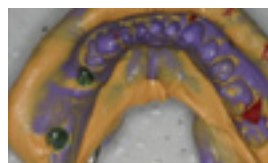

Obtain an impression.



Place the proper diameter implant lab analogs to the copings.



Reposition the copings on the impression body.

## Neo PickCap Impression Kit Tray





Product Name	PCKITTRAY
--------------	-----------

## Neo PickCap Impression Kit Cover







Product Name	PCKITCOVER
--------------	------------

## PickCap Impression Cap


	Product	Product Name	Note
Small Cap		PCISC01	Coping Ø3.5 ~ Ø4.5
Large Cap		PCILC01	Coping Ø5.2 ~ Ø9.0









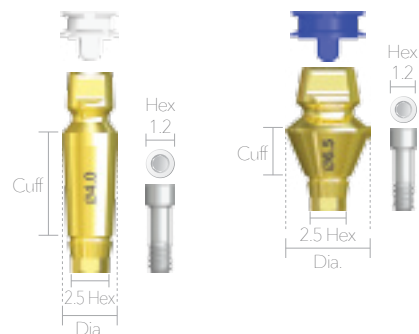

## Positioner & 1.2 Hex Driver





	Product	Product Name	Note
Positioner		AP165N	5mm
		HD1215S	15mm
1.2 Hex Driver		HD1220S	20mm
		HD1225S	25mm

## IS PickCap Impression Coping



Diameter	Small Cap		
	Ø3.5	Ø4.0	Ø4.5
	3mm	3mm	3mm
			
	5mm	5mm	5mm
			
Cuff (mm)	8mm	8mm	8mm
			
	10mm	10mm	10mm
			

Diameter	Large Cap				
	Ø5.2	Ø5.7	Ø6.5	Ø8.0	Ø9.0
	2mm	2mm	2mm	3mm	3mm
					
	3mm	3mm	3mm	4mm	4mm
					
Cuff (mm)	4mm	4mm	4mm		
					
	5mm	5mm	5mm		
					
	6mm	6mm	6mm		
					

## Pincette



Product Name	PCIPC
--------------	-------

Neobiotech Co.,Ltd

10F, 36, Digitalro-27Gil, Guro-Gu, Seoul, Korea, 08381

Tel 82-2-582-2885, Fax 82-2-582-2883 www.neobiotech.com / Made in Korea

**Neo  
Biotech**  
Satisfaction to Dentists